

The logo for 'alyors' features a stylized green icon of a person with arms raised inside a circular arrow, followed by the word 'alyors' in a bold, lowercase sans-serif font.

alyors

The IYON logo consists of the word 'IYON' in a bold, white, sans-serif font with a red swoosh above it, followed by a registered trademark symbol. Below it, the words 'PRODUCT CATALOG' and the website 'www.iyonmedical.com' are written in a smaller, white, sans-serif font.

IYON[®]
PRODUCT CATALOG
www.iyonmedical.com

IYON INTERNATIONAL PRODUCT CATEGORIES

Insulin Pen Needle



Nasal Care



Skin and Oral Care



Saline Solutions

Sinus Rinse Kit

Isotonic Ocean Water

Hypertonic Ocean Water

Baby Nasal Aspirator &
Disposable Tips

Lice Spray

Tea Tree Oil Shampoo

Canker and Mouth Ulcer
Solution

Glycerin

Iyon Medical was founded in 2008 to produce private label medical products for the market. Iyon Medical has an enclosed area of 13,500 square meters and a clean room of 2,000 square meters.

Thanks to its successful R&D activities, the company is proud to be the first domestic manufacturer of insulin pen needles and seawater by establishing its production facility in Turkey.

Iyon's production facility meets the standards of ISO and CE. The plant can operate in different work shifts to meet all future production needs of suppliers. Since all production machines are equipped with the latest technological automation system, products can be produced without being touched. All product components reach the plant in the form of raw material and are delivered to the end-user as a finished product.

Iyon products meet the standards of ISO and CE and have their registered trademarks and quality certificates. The products can not be placed on the market until they have successfully passed all the linear quality controls required by the Quality System, which is managed under the ISO 13485 and ISO 9001 standards.

Iyon Medical has created a fair and high-quality competitive environment by expanding its product portfolio to more than 12 different types of consumables and 250 different brands.

Based on its quality policy, Iyon Medical strives for the highest quality in every product to ensure customer satisfaction.

As Iyon, we have been investing in health since 2008.

Insulin Pen Needles



IYON Fine


Sterile (ethylene oxide), Non- toxic,
Non-pyrogenic, Hypodermic,
Suitable for single use.

- Painless injection and comfortable use
- Triple Beveled
- Silicone coated
- Produced by ultra-thin wall technology
- Compatible with all insulin pens

Sizes

- 31 G (0,25mm) x 4mm
- 31 G (0,25mm) x 5mm
- 31 G (0,25mm) x 6mm
- 31 G (0,25mm) x 7mm
- 31 G (0,25mm) x 8mm
- 32 G (0,25mm) x 4mm



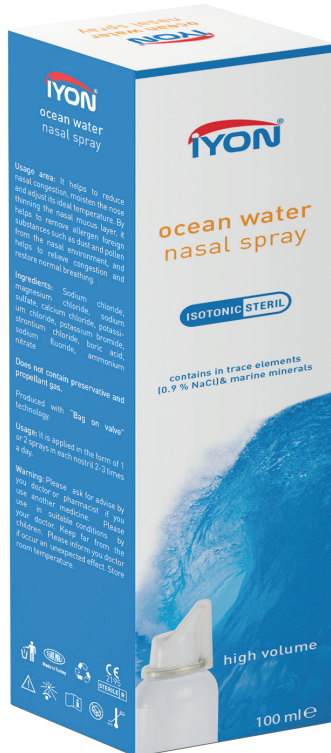

Isotonic Ocean Water Nasal Spray

100 percent Natural Sea Salt-based nasal spray, rich in marine minerals well known for their therapeutic properties. Enhanced with traced minerals such as strontium. Unique formulation with natural ingredients, minerals, and elements, Isotonic seawater content (0.9 % NaCl) provides decongestion of the nose and limits infections by decreasing germs.

IYON Nasal Ocean Water Spray has a unique nozzle that disperses the solution in fine droplets further up the nose. Does not contain preservative and propellant gas. Produced with “Bag on valve” technology.

Ingredients

Sodium chloride, magnesium chloride, sodium sulfate, calcium chloride, potassium chloride, potassium bromide, strontium chloride, boric acid, sodium fluoride

Direction

- Insert the nozzle into one nostril. Press briefly without tilting your or your head backward or sniffing. Blow your nose after use and repeat in other nostril. Pump 2 times into each nostril 3 times a day or as recommended by your doctor.

Uses

- By thinning the nasal mucus layer, helps to remove allergen foreign substances such as dust and pollen from the nasal environment and helps to relieve congestion and restore normal breathing.
- The use of isotonic ocean water spray contributes to accelerating recovery in flu/cold, sinusitis, and rhinitis.

Nasal Care



Hypertonic Ocean Water Nasal Spray

100 percent Natural Sea Salt-based nasal spray, rich in marine minerals well known for their therapeutic properties. Enhanced with traced minerals such as strontium. Unique formulation with natural ingredients, minerals, and elements. Hypertonic seawater content (1.8 % NaCl) provides decongestion of the nose and limits infections by decreasing germs.

IYON Nasal Ocean Water Nasal Spray has a unique nozzle that disperses the solution in fine droplets further up the nose. Does not contain preservative and propellant gas. Produced with “Bag on valve” technology.

Ingredients

Sodium chloride, magnesium chloride, sodium sulfate, calcium chloride, potassium chloride, potassium bromide, strontium chloride, boric acid, sodium fluoride

Direction

- Insert the nozzle into one nostril. Press briefly without tilting your or your head backward or sniffing. Blow your nose after use and repeat in other nostril. Pump 2 times into each nostril 3 times a day or as recommended by your doctor.

Uses

- It helps to reduce nasal congestion.
- It helps to remove allergen foreign substances such as dust and pollen from the nasal environment and restore normal breathing.
- The use of hypertonic ocean water spray contributes to accelerating recovery in flu/cold, sinusitis, and rhinitis.



The IYON logo features the brand name in a bold, blue, sans-serif font. A red swoosh underline is positioned above the letters 'Y' and 'O'. A registered trademark symbol (®) is located to the upper right of the 'N'. The logo is set against a white background that is part of a grey arrow-shaped graphic pointing to the right.

Nasal Care Saline Solution



Spray 25 ml / For nasal use

Sterile and does not contain preservatives and additives

Ingredients

Purified water,
Sodium Chloride (0.9%)

Uses

Helps moisturize dry nasal passage and relieve dry, crusty, and inflamed nasal membranes due to cold and allergies and is an effective nasal wash for sinuses.

Direction

For nasal use only
Use 2 to 4 times per day.
Pump 2 times (or apply 3 sprays) into each nostril.
Opened vials are disposable as they do not contain preservatives.

Nasal Care

Saline Solution



Drop 25 ml / For nasal use

Sterile and does not contain preservatives and additives

Ingredients

Purified water,
Sodium Chloride (0.9%)

Uses

Helps moisturize dry nasal passage and relieve dry, crusty, and inflamed nasal membranes due to cold and allergies and is an effective nasal wash for sinusitis.

Direction

For nasal use only
Use 2 to 4 times per day.
Pump 2 times (or apply 3 drops) into each nostril.
Opened vials are disposable as they do not contain preservatives.





Nasal Care Saline Solution



5 ml x 10 Vial / For nasal use

Sterile and does not contain preservatives and additives

Ingredients

Purified water,
Sodium Chloride (0.9%)

Uses

Helps moisturize dry nasal passage and relieve dry, crusty, and inflamed nasal membranes due to cold and allergies and is an effective nasal wash for sinuses.

Direction

For nasal use only
Use 2 to 4 times per day.
Pump 2 times (or apply 3 drops) into each nostril.
Opened vials are disposable as they do not contain preservatives.

Nasal Care Saline Solution



5 ml x 20 Vial / For nasal use

Sterile and does not contain preservatives and additives

Ingredients

Purified water,
Sodium Chloride (0.9%)

Uses

Helps moisturize dry nasal passage and relieve dry, crusty, and inflamed nasal membranes due to cold and allergies and is an effective nasal wash for sinuses.

Direction

For nasal use only
Use 2 to 4 times per day.
Pump 2 times (or apply 3 drops) into each nostril.
Opened vials are disposable as they do not contain preservatives.



Nasal Care

Nasal Wash



Sinus Rinse Kit Adults & Pediatr

IYON Nasal Wash relieves nasal symptoms and congestion caused by the flu, allergies, and sinusitis. Regular nasal washing removes germs, viruses, and allergens from the nose.

Ingredients

- 1 application bottle with 240 ml volume
- 20 packets (Sodium Bicarbonate 700 mg, Sodium Chloride 2300 mg)
- With 4% sodium bicarbonate and 96% sodium chloride (0.09% sodium chloride and 0.0375% bicarbonate in 240 cc) contained in the sachet, it is well tolerated with a balanced pH value compatible with the nasal mucosa.

Uses

- For daily nasal cleaning
- Nasal congestion caused by flu/cold and allergy
- Acute or Chronic Sinusitis
- Nasal dryness
- After nose and sinus surgeries

Nasal Care

Nasal Wash

The IYON logo features the brand name in a bold, blue, sans-serif font. A red swoosh underline is positioned above the letters 'Y' and 'O'. A registered trademark symbol (®) is located to the upper right of the 'N'.

Direction

1. After washing your hands, open the cap of the application bottle and fill it with 240 ml of warm water for adults or 120 ml for pediatric use.
2. After opening one of the mixture sachets in the box, pour it all into the application bottle. Make sure that the cap of the application bottle is tightly tightened and shake the application bottle to obtain the mixture.
3. Tilt your head straight down. Keeping your mouth open, place cap the application bottle on your nose.
4. Squeeze the application bottle lightly by hand. You can apply this application to the same nostril several times. For the application to be sufficient, you need to apply 1/4 or half of the bottle.
5. Repeat the same steps on your other nostril.
6. After the application is finished, quickly exhale the solution remaining in the nose and discharge it from the nose and throat.





Nasal Aspirator

Our Nasal Aspirator is designed to remove extra mucus from your baby's nose, helping him to breathe comfortably. Your baby will be able to sleep, eat, and play more peacefully without the uncomfortable sensation of nasal blockage, which prevents him from breathing easily. It is critical to clear the airways in newborns! The aspirator tips are made of phthalate-free, latex-free polypropylene and are designed for single-use to reduce the risk of infection. What's in the box: IYON Baby Nasal Aspirator consists of a flexible PVC tube, mouthpiece, a rigid body to keep the aspirator stable, and a transparent anatomical tip with a sponge filter designed for the baby's nostril.

Direction

1. Attach the disposable tip to the aspirator's body.
2. Take the mouthpiece in your mouth and insert gently the tip of the aspirator into your baby's nostrils.
3. Aspirate the mucus layer from the nose using the mouthpiece.
4. After use, remove the disposable tip and throw it away.
5. To facilitate the cleaning, saline or ocean water solution can be used before the procedure.

Nasal Care Aspirator Tips



Nasal Aspirator tips

Our Nasal Aspirator is designed to remove extra mucus from your baby's nose, helping him to breathe comfortably. Your baby will be able to sleep, eat, and play more peacefully without the uncomfortable sensation of nasal blockage, which prevents him from breathing easily. It is critical to clear the airways in newborns! The aspirator tips are made of phthalate-free, latex-free polypropylene and are designed for single-use to reduce the risk of infection.

What's in the box: IYON Baby Nasal Aspirator consists of a flexible PVC tube, mouthpiece, a rigid body to keep the aspirator stable, and a transparent anatomical tip with a sponge filter designed for the baby's nostril.

Direction

1. Attach the disposable tip to the aspirator's body.
2. Take the mouthpiece in your mouth and insert gently the tip of the aspirator into your baby's nostrils.
3. Aspirate the mucus layer from the nose using the mouthpiece.
4. After use, remove the disposable tip and throw it away.
5. To facilitate the cleaning, saline or ocean water solution can be used before the procedure.



Lice Spray

Lice Spray is used to cleanse the scalp and hair from lice and eggs by cutting off the contact of the lice with the air and providing a physical effect.

- Easy to use
- Odorless – colorless
- Insecticide free
- The most effective and safe way to get rid of lice and eggs
- Suitable for adults and children over 6 months old.
- Consult your healthcare specialist to use while pregnant or lactating.

Ingredients

Dimethicone, Cyclopentasiloxane

Direction

1. Comb the hair with the comb included in the box.
2. Apply spray to dry hair. Make sure that it does not come into contact with the face and eyes during application.
3. Apply by massaging the product to ensure that it is evenly distributed over the entire scalp.
4. Let the hair dry naturally and comb the entire hair after 30 minutes.
5. Wash the hair with your normal shampoo 6-8 hours after the application.
6. For eggs that cannot be cleaned in the first application, reapply after 7 days.

Skin & Oral Care Products



Tea Tree Oil Shampoo

Tea Tree oil shampoo is helped to protect your hair. After usage of the lice spray it could be used.


- Easy to use
- Suitable for adults and children over 6 months old.
- Consult your healthcare specialist to use while pregnant or lactating.

Ingredients

Aqua,Sodium Laureth Sulfate, Cocamide Betaine,Disodium Laureth Sulfosuccinate,Cocamide Dea,Polyquaternium-7, Melaleuca Alternifolia Leaf Oil,Piroctone Olamine,Citric Acid, Benzyl Alcohol, Hydroxyisohexyl 3-Cyclohexene Carboxaldehyde, Alpha-Isomethyl Ionone, Linalool, Methylchloroisothiazolinone, Methylisothiazolinone

Direction

Apply Tea Tree Oil Shampoo to wet hair and scalp.Leave shampoo for 3-5 minutes on the hair and then rinse.Repeat the shampoo application after seven days.

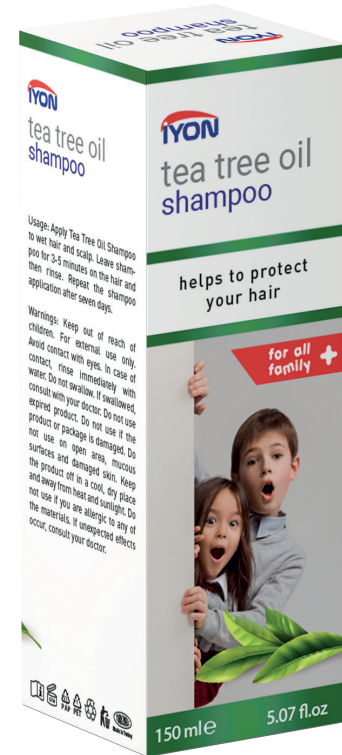



Canker Sore And Mouth Ulcer Instant Pain Relief

Aft Stick is used to relieve immediate canker sore and mouth ulcers pain.

It promotes spontaneous healing by creating a protective barrier over the ulcer.

Ingredients

- Hydroxypropyl methylcellulose, sulfonic acid, and sulfuric acid mixture
- 20 application swabs.

Direction

1. For each application, wipe the canker sore area with a cotton swab until it dries.
2. Then dip the other side of the stick into the bottle and tap the bottle rim once and apply to the canker sore area. Wait for a while and turn the litter around its axis.
3. You can apply 2-3 applications to the same area.

Skin & Oral Care Products



Glycerine


- Easy to use in a dropper bottle.
- Glycerine opens the blockage in the ear and facilitates the removal of wax.
- It has a moisturizing and softening effect on the skin.

Ingredients

30 gr 100% Pure Natural Glycerine.

Direction

- Lie on your side and in a straight position. 2 drops of ear drops are instilled. After waiting for a minute, get up and tilt the head in the opposite direction.
- Then the ear can be dried externally.

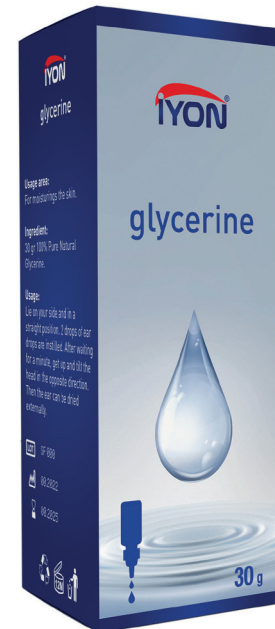

 alyors

 IYON[®]

Iyon Saglik Medikal Kozmetik Ith. Ihr. San. Tic. Ltd. Sti.
Osmangazi Mahallesi Battalgazi Caddesi No: 15
Sancaktepe/Istanbul Turkey Phone: +90 216 365 38 83
bilgi@iyonmedical.com www.iyonmedical.com